

# SALVIN® AlloSpark-GF™

Supplemental Allograft Derived Proteins

## Product Preparation Guide



**Step 1:** Using sterile technique, open the outer pouch and introduce the inner pouch and its contents into the sterile field.



**Step 2:** Carefully remove the red cap from the vial.



**Step 3:** Inject sterile water or sterile saline with a needle and syringe\* through the rubber stopper.\*\*




**Step 4:** Gently swirl and invert to ensure adequate mixing.



**Step 5:** Remove the metal ring by pulling up and out; then remove the rubber stopper from the vial.



**Step 6:** Remove the liquid from the vial using the needle and syringe.

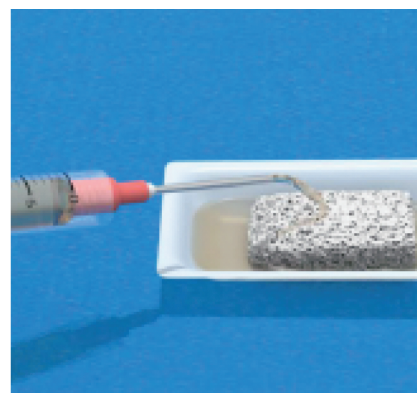

**Step 7:** Evenly distribute the liquid onto the desired scaffold.



**Step 8:** Allow a minimum of 15 minutes for growth factors to bind to the scaffold.